Système de communication Communication system

Kit système de communication Communication system kit

FR

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Table des matières

Manual EN	15
1. Introduction	16
1.1 Purpose	16
1.2 Targeted audience	16
1.3 Alert	16
1.4 Icons	16
1.4.1 Icons et notes	16
1.4.2 Attention, warning and danger icons	16
1.5 Conversion tables	17
2. Description	18
2.1 The range	18
2.2 Content	18
2.3 ST-DUO2	18
2.4 Safety note	18
2.5 compliance and autorisation	18
3. Power supply	19
4. Installation	20
4.1 Connect the microphone to the transmitter	20
4.2 Connect the receiver to the receiver	20
4.3 Positioning the device on the body	20
5. Operation	21
6. Use	22
6.1 Meeting	22
6.2 Conservation	22
6.3 Events	22
6.4 Tv	22
6.5 On the move	22
7. specifications	23

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Communication system set

EN

Installation and user manual
ST-DUO2

1. Introduction

1.1 Purpose

The Installation and Operation Manual provides the necessary information for installing, configuring and using an ST-DUO2.

1.2 Targeted audience

The Installation and Operation Manual is intended for installers and users of the ST-DUO Kit.

1.3 Alerts



This manual discusses four types of alerts. The type of alert is closely related to the effect that may occur if the alert is not observed. These alerts, ranked in ascending order of severity, are the following:

- **Note**
Additional information. Generally, the non-observance of a Note type alert does not result in any material or bodily injury.
- **Attention**
Failure to observe a caution alert may result in property damage.
- **Warning**
Non-compliance with a type alert Warning may result in serious personal injury and property damage.
- **Danger**
Failure to observe a danger alert may result in death.

1.4 Icons




1.4.1 Icons and notes

Icons used with notes provide additional information about it. See the following examples: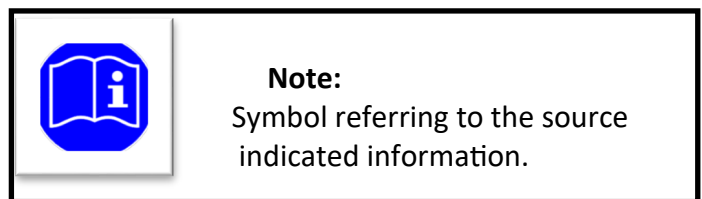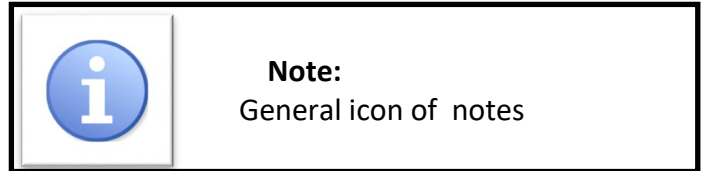

1.4.2 Attention, warning and danger icons

The icons used in combination with Attention, Warning and Danger indicate the type of risk present. See the following examples:

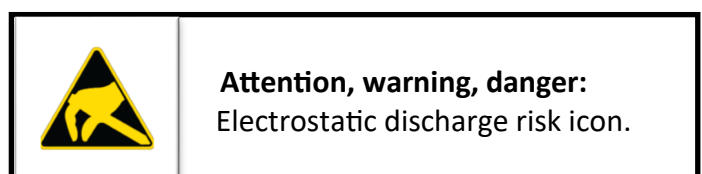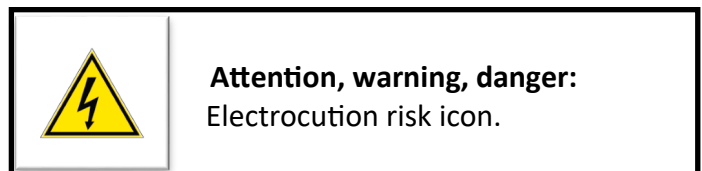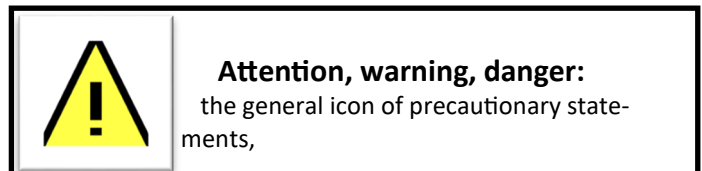1.5 Conversion tables

In this manual, SI units are used to express lengths, masses, temperatures etc.

These can be converted to non-metric units using the following information.

Table 1: length units conversion

25,40 mm = 25,4 mm	1 mm = 1,00000
25,40 mm = 2,54 cm	1 cm = 0,3937 po
30,48 cm = 0,3048 m	1 m = 3,281 pd
1 ml = 1,609 km	1 km = 0,622 ml

Table 2: Mass units conversion

1 lb = 0,4536 kg	1 kg = 2,2046 lb
------------------	------------------

Table 3: Pressure units conversion

1 psi = 68,95 hPa	1 hPa = 0,0145 psi
-------------------	--------------------

Note:

1 hPa = 1 mbar

Table 4: temperature units conversion

$^{\circ}F = 9/5. (^{\circ}C + 32)$	$^{\circ}C = 5/9. (^{\circ}F - 32)$
--------------------------------------	--------------------------------------

2. Description

ST-DUO2 is a wireless communication system for people with hearing loss. It offers a high fidelity listening quality. This audio transmission device provides clear and precise communication in complicated listening environments such as an amphitheater, meeting room or around a family meal.

The system is compatible with hearing aids and cochlear implants but also offers a qualitative solution for people who are not paired.

The nomad kit can be transported easily.

2.1 The range

The ST-DUO2 kit is part of the Opus 10 UHF system range. Opus 10 offers audio accessibility solutions for public-facing establishments. It is a range of combined systems to create sound systems and public audio accessibility adapting to all situations.

The Opus 10 range includes:

- Transmitters and receivers
- Accesories.

The various elements are thought to complement each other, thanks to acoustic, technological or mechanical characteristics ensuring their compatibility.

2.2 content

Package content:

- A transmitter with microphone
- A receiver with a helmet or a neck buckle
- Adapters (TV, computers, ...)
- Transport suitcase

2.3 ST-DUO2

Combined with one or more OP-10R receivers, the OP-10T transmitter forms a mobile wireless system for speech transmission, as for example guided tours or speeches in multiple languages. For wireless transmission, 16 channels in the frequency range 863 - 865 MHz are available, of which 3 can be used simultaneously at most without interfering with each other. The range of transmission depends on the configuration of the places of use and can be up to 60 m.

2.4 Security

The device meets all the necessary directives of the European Union and therefore bears the symbol.

The device is intended for indoor use only. Protect it from moisture and heat (admissible ambient temperature 0 - 40 ° C).

For cleaning, use a dry, soft cloth, under no circumstances chemicals or water.

We do not accept any liability for personal injury or property damage if the device is used for a purpose other than that for which it was designed, if it is not used properly. In addition, if repaired by an authorized person, the warranty will lapse.

2.5 Compliance and autorisation

OPUS TECHNOLOGIES hereby declares that the OP-10T transmitter is in compliance with the basic requirements and regulations of Directive 1999/5 / EC. The declaration of conformity may be requested.

The system is authorized in the European Union and the countries of the EA and requires no declaration.

3. Power supply

The transmitter / receiver is powered by a Lithium-ion battery. Before first use, the battery must be fully charged. To load it, the following chargers are available:

- OP-10CH2 charging station for charging batteries of up to two devices simultaneously.
- OP-10CH18 transport case to charge the rechargeable batteries of up to 18 devices simultaneously.
- Transport case on wheels OP-10CH36 to charge batteries of up to 36 devices simultaneously.

In case of prolonged non-use, it should be recharged every three months. Any defective battery must be replaced by a technician as the device must be open. A suitable spare battery of the OPUS Technologies brand is referenced OP-3.7VB.


Do not dispose of defective accumulators in the household trash, put them in a specific container.

4. Installation

4.1 Connecting the microphone to the transmitter


If you want to use the headset microphone instead of the internal microphone, connect it to the MIC INPUT jack (1).

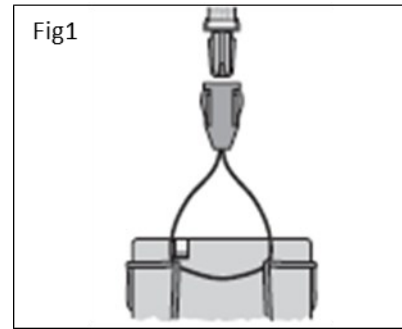The internal microphone is then disconnected. If necessary, place an anti-pop windshield on the microphone capsule. Place the earplug stirrup on the left ear so that the lobe support leans back against the lobe. You can attach the micro cord to clothes with the clip.

4.2 Connecting the receiver to the receiver


To pad the earphone, it can be covered with the foam cover: put the earphone, printed face back, Position the cover around the earpiece and put over the second envelope.  The mark  represents the output side of the sound. Connect the earphone to the jack  (1) and put it on the flag of the ear.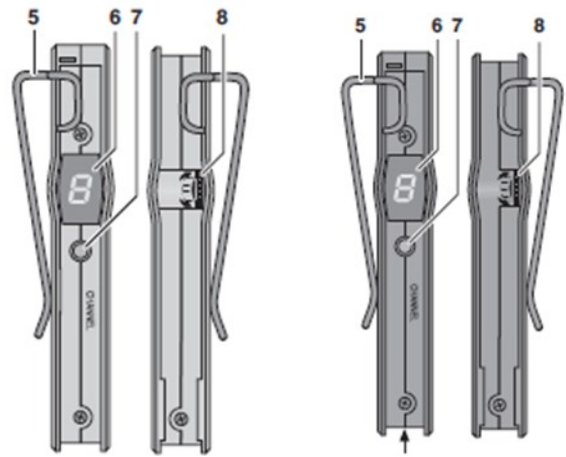


If the sound is to be transmitted to a hearing aid, the TR-iL induction loop (available as an Opus Technologies accessory) can be connected at place of the earpiece. It is placed around the neck and transmits the sound via a magnetic field to the hearing instruments with the T position.

4.3 Positioning the device on the body

The transmitter / receiver can be attached to a garment via its stirrup (5) [eg at the belt, on a jean's pocket] or worn around the neck: pull the strap fastening loop via the stirrup; by moving the button on the shoulder strap, set the desired length then put the shoulder strap around the neck. By unlocking the attachment system, (left diagram), you can easily remove the device and replace it with another one. Fig1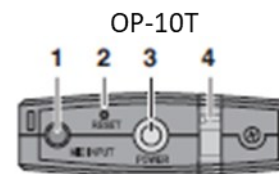

Microphone transmitter for transmitting audio

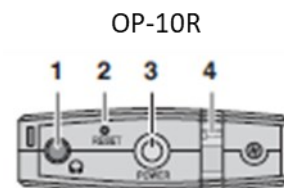Listening receiver

5. Operation

1) To turn on the system devices, press the POWER button (3). The operation indicator (4) lights up in green. If it glows red, the battery is almost empty, it should be recharged as quickly as possible.

After each key press, the display (6) indicates for approximately 9 seconds the transmission channel set, then the indication disappears. By briefly pressing a button, the display can be reactivated for about 9 seconds.

2) Select the transmission channel among the channels available at. The transmitter and receiver (receivers) of the system must be set to the same channel. To adjust the channel, press and hold CHANNEL (7) until a dot on the display next to the channel indication flashes. Then press the CHANNEL key until what the desired channel is displayed. The selected channel is memorized if approximately 6 seconds after the last CHANNEL key is pressed, the dot on the display disappears.

3) On the transmitter, talk to the internal microphone (9) or the connected headset. With setting (8), you can adjust the volume on the transmitter for the audio signal output and on the receiver you can adjust the playback volume.

Caution: Never adjust the volume very high. Too much volume can cause long-term hearing problems!

4) If reception is poor or disturbed, check if:

- the transmission is better on another channel.
- the distance between the transmitter and the receiver is too large
- There are obstacles between the devices that can disrupt the radio signal.

Up to 3 OP-10 systems emitting on different channels can be operated in parallel and without interference. In parallel operation of multiple systems, be careful to maintain enough distance between each system.

5) To reset a device in case of malfunctions, press the RESET key (2) with a thin object, for example a metal pin. The unit turns off and can be turned back on.

6) To switch off, press and hold the POWER button until the operation light goes off.

Loading devices:

1) Connect the mains unit to the socket on the back of the charging station and to a 230 V ~ / 50 Hz socket. The station is then switched on. As long as a charging compartment is empty, its operation indicator flashes.

2) Place the transmitter / receiver in a charging compartment, load contacts facing down. When positioning it, be sure to push the unit all the way into the compartment so that it is secure. During charging, the charge compartment operation indicator lights continuously.

3) When the battery is fully charged, the corresponding charge lamp turns off. The device can then be removed from the charging compartment.

4) After charging, unplug the power supply to avoid unnecessary power consumption.

Note:

If a charge indicator flashes during charging, the charging station performs a self-test to protect the battery

6. Use

The ST-DUO2 can be used in different ways as described below:

6.1 Meeting

For a conversation around a table place the OP-10T transmitter in the center of the board. The integrated microphone will capture the voices of people placed 1 m from the box. For better sound capture it is possible to use a wired microphone putted on the table (option).

6.2 Conversation (example: restaurant)

Place the OP-10T transmitter around the other person's neck or waistband and plug in the lapel microphone provided in the kit on the OP-10T.

6.3 Event (course, lecture, etc)

Place the OP-10T transmitter near the speaker or speakers (example on the teacher's desk) or give the transmitter OP-10T with a lapel microphone so that the person installs it at the beginning of the event.

6.4 Connected to TV

Note:

It is possible to use multiple OP-10R receivers with an OP-10T transmitter.

Thanks to the cord provided in the pack, connect the OP-10T to the audio output of your television. For better listening quality, use a TO-Slink converter that will allow you to be connected to the digital output of your TV. In many cases it will also not cut the sound of the speakers.

6.5 On the move

Hold the OP-10T transmitter on the microphone side and extend it to the other party to pick up his voice (example: contact a store salesman).

6. Specifications

Frequency range

Radio	863-865 MHz
Audio bandwidth	40-18 KHz
Transmission power	< 10mW
Internal transmitter microphone	Micro electret (cardioid)

Accessories

Transmitter	Microphone electret headset (omnidirectional) with cable clip and 2 anti-pop cups, neck buckle
Receiver	Dynamic mono headset with padding, neck buckle
Power supply	Accu Lithium-ion, 3.7 V/1300 mAh
Dimensions, weight	54 x 99 x 17 mm, 80g

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